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CONTENTS

| | | | | | |
|----------|---|----------|----------|---|-----------|
| 1 | Product Description | 3 | 5 | Operation | 9 |
| 1.1 | Design Principle..... | 3 | 5.1 | Starting Operation | 9 |
| 1.2 | Intended Purpose | 3 | 5.2 | Shut-Down Actions | 9 |
| 1.3 | Used Standard | 4 | 5.3 | Cleaning and Desinfection | 12 |
| 2 | Information on Instruction for Use | 4 | 6 | Maintenance..... | 13 |
| 2.1 | Reducer Types | 4 | 6.1 | Warranty, Customer Service and Dispatch | 14 |
| 2.2 | Referred Persons | 4 | 6.2 | UMDNS-Code | 14 |
| 2.3 | Improvement | 5 | 6.3 | Trouble Shooting..... | 15 |
| 2.4 | Symbols..... | 6 | 7 | Appendix | 16 |
| 3 | Safety..... | 6 | 7.1 | Technical Data..... | 16 |
| 3.1 | Safety Instructions | 6 | 7.2 | Flowmeter Options | 17 |
| 3.2 | Rules & Regulations | 7 | 7.3 | Ordering information | 18 |
| 4 | Installation | 7 | 7.4 | Options | 19 |
| 4.1 | Transport and Packaging | 7 | | | |
| 4.2 | Preparation | 7 | | | |

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1 PRODUCT DESCRIPTION

1.1 Design Principle

The TESCOM EUROPE Flowmeter is an armature, which the gasflow control becomes measured. The Flow is determined with a variable area flowmeter. The gas flows from the bottom to the top against the weightforce. The tube in which the suspense body is located is conical. The height of the suspense body is dependence of the flow. The flow will be controlled through a needle valve after the flowmeter.

1.2 Intended Purpose

Flowmeter to control flow of oxygen or medical air for inhalation or insufflation, in conjunction with the accessories need for operation.

Only for patients with spontaneous breathing.

- to be connected to central gas supply either for:
- oxygen O₂
- air

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-
- for supply of medical devices acc. to technical data of the identification plate or instruction for use.

1.3 Used Standard

CE designation acc. to EG-Standards 93/42/EEC class IIa

Flowmeter is acc. to:

DIN EN 13220

DIN 13260

2 INFORMATION ON INSTRUCTION FOR USE

2.1 Reducer Types

This instruction for use is valid for the TESCOM EUROPE flowmeter.

2.2 Referred Persons

These instructions for use refer to all persons who install, operate and maintain the above mentioned units. It presupposes, that the persons are familiar with the general rules and safety regulations concerning handling gas and gas fittings. Do not try to operate units, if you have not read and understood Chapter "Safety" and Chapter "Operation" at least.

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The reducer is only destined for the described intended purpose.

Under certain conditions TESCOM EUROPE GMBH & CO. KG permits the maintenance via persons, who are not employed by TESCOM EUROPE GMBH & CO. KG. The conditions are as follows, the skilled personnel must be trained and the instructions and information given in Chapter "Maintenance" must be observed.

Make sure that the operating personnel is informed about these instructions! Further copies are available on request.

2.3 Improvement

If you have suggestions for improvement concerning the product or the instructions for use, we would be glad to consider them. Our products as well as the instructions for use are steadily in development. You will find the address and telephone number of TESCOM EUROPE GMBH & CO. KG on the last page of these instructions for use.

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2.4 Symbols

This symbol points to special data and/or rules and prohibitions concerning damage precaution. These instructions are important for working safety!

This symbol stands in front of especially important instructions for regulation observation or in case of property damage danger.

This symbol means free from oil and grease.

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3 SAFETY

3.1 Safety Instructions

Reducer and connected devices must be kept clean and free from oil and grease! Oil and grease may react viciously when brought into contact with certain compressed gases. Danger of explosion!

Prior to connecting the reducer must be ensured that the supplied gas is identical with the designation on the reducer.

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O₂ Handling: Oxygen improves combustion when brought into contact with flammable materials. Ensure sufficient ventilation. No smoking or open fire.

3.2 Rules & Regulations

The national rules and regulations concerning the handling of

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4 INSTALLATION

4.1 Transport and Packaging

The TESCOM EUROPE Flowmeter may only be shipped (e.g. dispatch to customer service) in an appropriate, stable packaging.

4.2 Preparation

Essential for the correct function of the TESCOM EUROPE Flowmeter is the cleanliness of plug and the purity, dryness and particle cleanliness of used gases.

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-
1. Check identification plate to ensure that the unit is suitable for the intended use (pressures, gas, connections). You will find both on the measuring tube.
 2. Remove protective caps. Use only dust free and dry gases.
 3. Measuring tube has to be installed vertical.

4. Prepare accessories in accordance with accompanying Instruction for use. Accessories with screw cap screw on flowmeter or connect to connecting hose.

All fittings must be clean and free from oil and grease!

Do not use any lubricants! Contaminates the reducer and may cause ignition with the use of O_2 .

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5 OPERATION

5.1 Starting Operation

1. Accessories needed for operation coupling to the outlet of the Flowmeter.
2. Turn handwheel anticlockwise. The floating part in the measuring tube must rise evenly.

 Read flow on upper rim of floating part.

Gas bursts induced by a quick opening of the valve may cause explosions!

3. Set required flow.

You will change the flow by turning the handwheel.

5.2 Shut-Down Actions

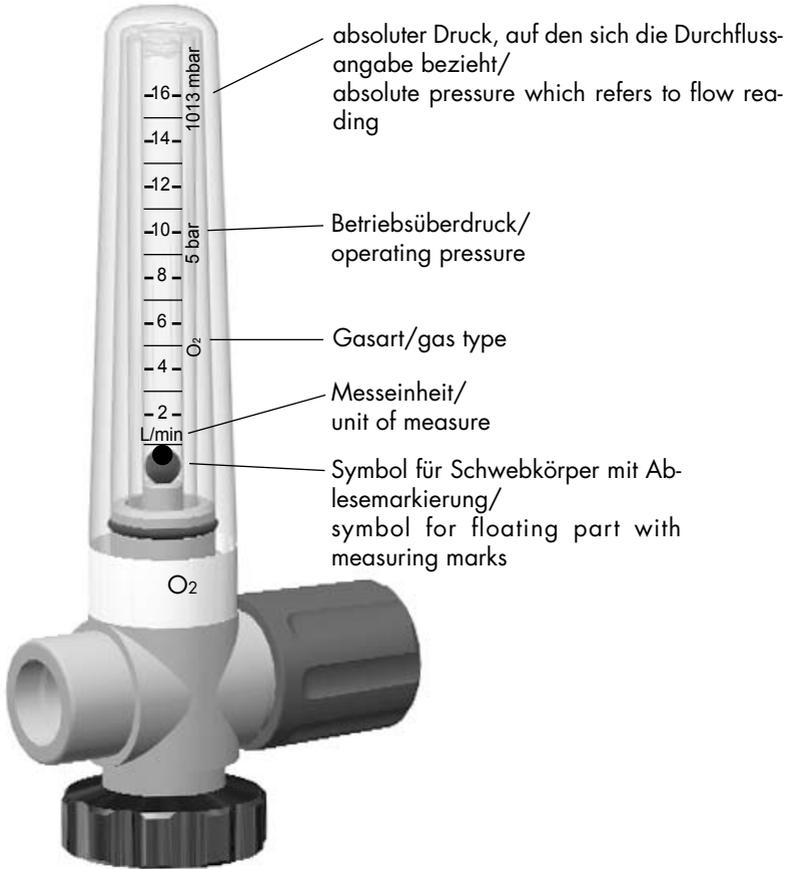
1. Turn handwheel clockwise.

Using only slight pressure, otherwise the seal may become damaged.

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5.3 Cleaning and Desinfection

Before cleaning or desinfection:

Plug inlet and outlet of Flowmeter with suitable caps, if reducer is not mounted to a device.

Cleaning

Clean Flowmeter surface with a disposable cloth.

Desinfection

Use common surface disinfectants. Follow manufacturers instructions.

For reasons of material compatibility, use disinfectants based on

- aldehydes,
- quaternary ammonium compounds.

The following agents may damage the materials and are not suitable:

- halogen-releasing compounds,
- strong organic acids,
- oxygen-releasing compounds.

For users in Germany, we recommend that only disinfectants on the current DGHM (DGHM: German Society for Hygiene and Microbiology) list be used. The DGHM list specifies the composition of each disinfectant.

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For countries in Germany, in which the DGHM list is unknown, the above disinfectant types are recommended.

The Flowmeter may not be immersed in any liquids and may not be sterilised!

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6 MAINTENANCE

The maintenance may only be performed by trained experts by using original TES-COM EUROPE parts!

Inspection every 6 months if a system has a normal duty cycle. The device is checked for outside damages and function.

The apparatus must be inspected and serviced every 3 years by trained service personnel.

More frequent maintenance may be necessary when the system is used under extreme conditions.

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6.1 Warranty, Customer Service and Dispatch

All products of TESCOM EUROPE GMBH & CO. KG are guaranteed to be free from defects in materials and workmanship for a period of 24 months. When operating with toxic or corrosive gases a reduced warranty of 6 months is valid.

For more information see "Warranty" in our terms of business in the delivery note.

See the manufacturers' address on the last page of these instructions for use.

Before return of equipment for repair please contact TESCOM EUROPE GMBH & CO. KG. In interest of the employees health we also need a declaration about the possibility of toxic or harmful media which may have contaminated the unit returned to us and which steps have been taken to avoid danger to persons (e.g. purging).

Observe the instructions in chapter "Installation" in the section "Transport and Packing".

6.2 UMDNS-Code

UMDNS-Code 11-748
Nomenclature for medical products.

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6.3 Trouble Shooting

| Fault | Cause | Remedy |
|---------------------------------------|-------------------------------|--------------------|
| Leakage between Flowmeter and outlet. | Damaged seal. | Replace seal. |
| Leak in metering valve. | Soiled or damaged valve seat. | Repair by experts. |
| Flow is not adjustable. | Damaged metering unit. | Repair by experts. |
| Nominal flow are not reached. | Blocked filter. | Repair by experts. |
| Leakage between flowmeter and outlet. | Damaged seal. | Replace seal. |

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(acc. to measuring tube) 2 to 16 l/min

4 to 32 l/min

Accuracy of flow rate at 5 bar ± 15 % (corresponding to smallest setting value)

(1013 hPa/20 °C)* ± 10 % (all other setting values)

Connections: see order no.

Weight: 0.35 kg

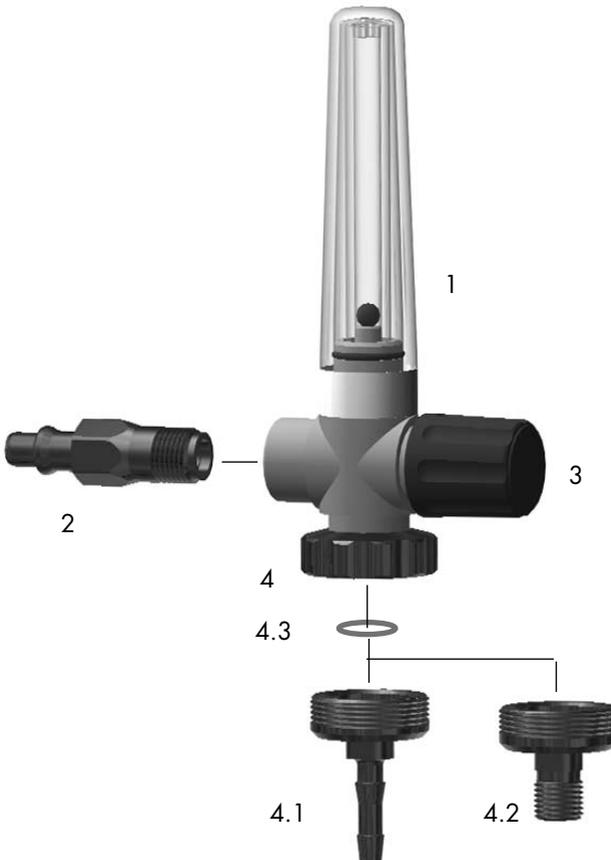
* If the flowmeter is used outside the specified technical data, this could affect flow accuracy.

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7.2 Flowmeter Options

- 1 Measuring tube with flow display
- 2 Inlet
plug for medical gas supply
- 3 Handwheel for flow adjustment
- 4 Outlet
- 4.1 Hose connector
- 4.2 Connector with screw thread
- 4.3 Seal

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- 1) See flowmeter options.
- 2) You will find the part no. on the label of the flowmeter.
- 3) Pack of one piece.

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X Acc. to different specifications of the catalog page.

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